anthogyr

PIECE A MAIN DROITE 1:1

STRAIGHT HANDPIECE 1:1

MANIPOLO DIRITTO RAPPORTO 1 : 1

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PIEZA DE MANO 1:1

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INSTRUCTIONS FOR USE

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INSTRUCCIONES DE USIO

PIECE A MAIN DROITE 1:1	3
STRAIGHT HANDPIECE 1:1	9
MANIPOLO DIRITTO RAPPORTO 1 : 1	15
HANDSTÜCKH RECHTS 1:1	21
PIEZA DE MANO 1:1	27

I. CONTENU

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Please wear
gloves

REMARKS

General indications
with no danger
to people or objects

Please read this !

Visual Examination

III. AREA OF APPLICATION

The handpiece is used for dental care, such as denture retouching, working on the dentine, cutting of tooth or parts of the tooth, denture cutting, any other form of use is forbidden and can cause danger.

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In accordance with these provisions, the handpiece must be used by someone with experience of dental medicine, for the application described, and in compliance with the provisions in effect in relation to the prevention of accidents at work, protection of employment and the instructions for use. Preparation and maintenance of this medical appliance have to be done exclusively by someone having training on prevention against infections, on self-protection and on protection of the patients.

In accordance with these provisions, the user must:

- Only use working instruments without defects,
- Only use the handpiece on units that are in accordance with the standard EN 60 601
- observe the principle of correct use,
- protect themselves and the patient or third party against any danger
- prevent all contamination by the product

IV. GENERAL SAFETY INFORMATION

Before use, check that the handpiece is not damaged and that no part of it is missing.

Use protective gloves, glasses and a mask.

When inserting or changing the rotary instrument, make sure that the handpiece is stationary.

After insertion of a rotary instrument, check that it is correctly held by a slight rotary movement.

Do not handle the rotary instrument during operation.

Only operate the locking catch when the motor is stopped.

The handpiece should be connected or disconnected to the motor when the motor is stationary.

In order to cool the treatment area, only an adapted and a correctly tempered cooling spray must be used.

Always make sure that the spray is sufficient and regular.

In the event of malfunction or damage, immediately stop using the instrument and inform your approved distributor or the manufacturer.

In the event of any questions about the equipment, contact either your approved distributor or the manufacturer.

No alteration or addition is to be made to the product without Anthogyr's express agreement.

V. TECHNICAL SPECIFICATIONS

Reference	4330D
Reduction / multiplication rate	1 : 1
Colour code	blue
Weight (g)	39 g
Motor connection standard	ISO 3964
Maximum motor speed (rpm)	40000
Maximum instrument torque (N.cm)	6 N.cm
Tool type according to ISO 1797-1	Type2
Diameter of tools (Ø mm) according to ISO 1797-1	2.35
Maximum length recommended by Anthogyr (mm)	45*
Maximum diameter of the active part of the tool recommended by Anthogyr (Ømm)	10*
Water spray output according to ISO 7785-2 (ml/min)	>50

* Suggested values. If using longer and wider diameter rotary instruments, the user is responsible for the correct choice of the operating conditions to prevent any risk to the patient or to a third person.

VI. USING THE HANDPIECE

The handpiece is supplied unlubricated and non-sterile.

Before first using it, the handpiece must be lubricated and sterilized (see "Hygiene and maintenance" paragraph).

Before use, check that the handpiece is not damaged or that no part of it is missing.

5.1. Connecting to the motor

WARNING : the handpiece should be connected to the motor only when the motor is stationary.

Lock the handpiece to the motor coupling. To do so, keep the motor and the contra angle on the same axis.

For certain motors, it is necessary to operate the side button to allow connection.

Pull the handpiece slightly before use in order to check that it is securely connected to the motor coupling.

Operating test > Operate the motor

NOTE : If you notice heating, irregularities, vibrations or abnormal sounds during the operation of the handpiece, immediately contact your approved technical department.

5.2. Disconnecting from the motor

WARNING : The handpiece should only be disconnected when the motor has stopped.

Remove the handpiece whilst keeping it in the motor axis.

For certain motors, it is necessary to operate the side button to disconnect the handpiece.

NOTE : In the event of prolonged non-use of the handpiece, do not leave it connected to the motor so as to avoid oil leakage into the motor. The motor could be damaged.

5.3. Connecting / disconnecting the rotary instrument

It is preferable to wear protective gloves for all tool handling. Cut risk.

Check the condition of the rotary instruments used and handle them cautiously and carefully.

It is essential that the motor is stopped when inserting or extracting the instrument.

Do not use worn instruments

Inserting and locking the rotary instrument:

- Turn the dark middle ring the "open" setting
- Insert the rotary instrument into the handpiece's head
- The depth of the instrument may vary. At least 12mm of the rotary instrument must be inserted.
- turn the dark middle ring to the "close" setting

Check that the rotary instrument is correctly held.

- Check that the rotary instrument is correctly held with a slight axial movement each time the tool is changed.

Removing the rotary instrument:

- turn the dark middle ring to the "open" setting
- withdraw the tool (rotary instrument)

VII. HYGIENE AND MAINTENANCE

Re-sterilization of reused medical appliances has to be done by someone who has been trained and is protected, and the regulations in force have to be adhered to. The re-sterilization protocol must be adapted to the infectious risk.

For each used product : please refer to the manufacturer's instructions. Respect in particular the concentrations, the exposure durations, the solutions' replacement and the products' lifetime. Never mix the products. Please dispose of products as per their instructions. Use only products aimed at surgical material maintenance.

Do not use antiseptics designed for mucous or skin. Use only products with alkaline or neutral Hp (between 2.5 and 9.0). Do not use products including Aldehydes, alcohol or other products that may fix protein.

Only use products which are compatible with stainless steel. (no chlorinated compound) and aluminium.

Do not immerse the handpiece in non approved solution.

Do not clean the handpiece in an ultrasonic tank.

Wear protective clothing.

Wear protective gloves, in order to avoid any risk of infection and wound. Moreover, motor shall not work during all handlings.

7.1. Cleaning

Clean the material immediately after the surgery.

Withdraw the handpiece from the motor.

Withdraw the rotary instrument (tool) from the handpiece. (see § 5.2.)

-brush carefully the handpiece with a soft brush under the water. (see § 5.2.)

-Dry the handpiece

-Clean the material by spraying or with cleaning paper soaked with bactericidal microbiologically controlled products (Hp between 2.5 and 9.0)

Whatever the products used, comply with the indicated acting time.

Do not use bactericidal products with alkaline or chlorinated compound.

Do not immerse the handpiece in bactericidal baths or in an ultrasonic tank.

Do not use sodium hypochlorite (bleach) : there is a high risk of corrosion.

Do not use saltwort : there is a high risk of corrosion.

7.2. cleaning of the sprays

Cautiously clean the spray nozzles using the metal wire Ref. DCB 9205 to remove dirt and deposits.

There is a risk of injury with the metal wire. It is important to wear protective gloves and glasses. If the nozzle remains clogged, contact the After-Sales Service or the approved technical department to replace it.

X To prevent any stagnation of the spray water in the contra angle duct, we recommend you blow air through the cooling liquid duct. Wear protective glasses and a mask when you do this.

7.3. Lubrication

This should be done before first use.

It also must be done before every sterilization.

Keep away from any heat or fire source.

Do not smoke!

Wear a protective mask.

Remove the rotary instrument (see §1.3.).

Insert the spray into the handpiece (Anthogyr reference : 1930). Make sure that the nozzle is the right one.

Cover the head of the handpiece with a small decontaminating cloth.

Put the head upside down.

Spray several times until the liquid coming out of the handpiece is clear.

Wipe the oil excess off with a rag or with a small cloth.

Warning : for lengthy operations, repeat every 30 minutes.

7.4. Operating test

This test has to be done prior to each sterilization.

Connect the handpiece to the motor ; put the handpiece head upside down. Slowly rotate the contra angle for 30 seconds X to get rid of the oil excess, and then rotate the contra angle faster for 30 seconds. Wipe the handpiece with a small cloth if oil has flown out.

If you notice heating, irregularities, vibrations or abnormal sounds during the operation of the contra angle, immediately contact your approved technical department.

7.5. Sterilization

The handpiece and its accessories must be **sterilized before the first use and after each use.**

Material sterilizable in autoclave only at 135°C - 2.13 bars (275°F – 30.88 psi) during a minimum of 20 minutes.

Class B autoclave is advised.

No other sterilizable process is allowed.

Read the instructions manual of the autoclave manufacturer.

Comply with the space between the bags and do not overload the autoclave.

-to be done prior to sterilization: first disinfection, cleaning, lubricating and testing.

-Withdraw the rotary instrument (tool) before sterilization (see § 5.3)

-Remove the spray nozzle from the handpiece.

-Check that the material has no rust and no crack. Check its working correctly. Check it is dry. If not, dry it with medical quality air under pressure.

-Use the sterilization bags suited for handpieces and for autoclave (norm NF EN 868). Put one handpiece per bag.

-To avoid any water retention, put the bag into the autoclave so that the hollow parts are upside down.

When autoclave has different working cycles, chose the one for medical material (minimum 135°C - 2.13 bars (275°F – 30.88 psi) for 18 minutes.

-After each sterilization, check there is no water left inside or outside the packaging. Check the indicator for the correct colour change.

-Keep the material in the sterilization bag, in a lightless dry space, free from sources of contamination. Follow the packaging manufacturer instructions.

-After its sterilization, material can be kept in its sterile packaging for a maximum of one month (not more). Label the material with the use-by date. If the product has not been used prior to the use-by date, re-clean and sterilize.

VIII. REPAIR

In the event of breakdown, please contact your approved distributor or the Anthogyr After-Sales Department directly.

All repairs must be carried out with parts and subassemblies approved by the manufacturer. Repairs must only be carried out by an approved distributor or by the After-Sales Service Department in the factory.

The complete handpiece must be returned in a sterile condition to the After-Sales Service Department. The handpiece must be sent together with a document stating the problem as well as the complete name and address of the dentist.

For any claims to be considered under the guarantee, please enclose a copy of the invoice or the delivery note with the handpiece.

ANTHOGYR
Service Après Vente
2237 avenue André Lasquin
74700 SALLANCHES FRANCE
Tel +33 (0)4 50 58 02 37
Fax +33 (0)4 50 93 78 60
Direct SAV : +33 (0)4 50 58 50 53

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X. GARANTEES

This handpiece is guaranteed for parts and labour against any manufacturing defect for a period of 12 months from the date of invoicing. This guarantee does not apply to parts that wear out in normal use and does not cover transport costs.

For claims to be considered under the guarantee, please enclose a copy of the invoice or the delivery note with the contra angle. Any modification or addition to the product without X Anthogyr's express agreement will nullify this guarantee.

The guarantee is invalid in the event that the technical instructions supplied with all our contra angles are not followed.

Anthogyr cannot be held responsible for damage and its consequences resulting or possibly resulting from normal wear and tear, incorrect use, cleaning or maintenance, non-compliance with the conditions for use or connection, scaling or corrosion, impurities in the air or water supply systems, or abnormal chemical or electrical effects or non-compliance with the Anthogyr instructions for use, maintenance or installation and any other instructions by the manufacturer.

XI. DISPOSAL

The material must be sterilized before being disposed. (see §VII)

According to current knowledge, this material does not contain noxious substance for the environment. Follow the law in force.

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